


The catalog of products for the visually impaired and hard of hearing Customer Service: 800-468-4789 9am-5pm Eastern Monday-Friday

# LS&S, LLC Item # 221111



Manual Instruction for Bone **Conduction Headphones** 

## Summary of button functions

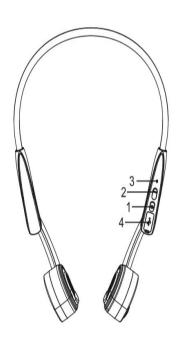



1.Power On/Off/Volume up

- 2. Volume down
- 3.LED indicator
- 4.USB charging port
- 5. Multifuction Button: Pause/Play/Answer Phone Call/ End Phone Call/ Next Track/
- Last Track/ Voice Assistant
- 6.Microphone

## Controls

On: When the headphone is in power-off state, press and hold power button for 3 seconds with blue LED flickering, reminding tone: power on. Off: When the headphone is in power-on state, press and hold power button for 5 seconds with red LED flickering once, reminding tone: power off.



The catalog of products for the visually impaired and hard of hearing Customer Service: 800-468-4789 9am-5pm Eastern Monday-Friday



- 1.Power On/Off/Volume up
- 2. Volume down
- 3.LED indicator
- 4.USB charging port
- 5.Multifuction Button: Pause/Play/Answer Phone Call/ End Phone Call/ Next Track/ Last Track/ Voice Assistant
- 6.Microphone

#### Controls

On: When the headphone is in power-off state, press and hold power button for 3 seconds with blue LED flickering, reminding tone: power on.
Off: When the headphone is in power-on state, press and hold power button for 5 seconds with red LED flickering once, reminding tone: power off.

Connect BT: When the headphone is in power-off state, press and hold it for 3 seconds to enter a pairing state with red and blue LEDs flickering and reminding tone: pairing. After the device (smart phone, tablet computer, etc.) discovers J20, press it to connect.

Connected successfully: When the headphone is successfully linked to the device, blue LED will flicker every 3 seconds with reminding tone: Connected

Pause/Play: When the music is on, single click the multi-function button to pause or play.

Answer/Hang-up: click multi-function button to answer when there is an incoming call. Click the multi-function button to end the on-going call. Vol+: Press the Vol+ once to turn up the volume. Vol-: Press the Vol- once to turn down the volume. Next Track: Double-click the multi-function button

skip to next track. **Last Track**: Triple-click the multi-function button skip to last track.

Assistant: Long press the multi-function button for 3 seconds to enter Voice Assistant model. Long press 3 seconds again to exit the Voice Assistant model Reset: After plugging the USB charging line, the headphone will be reset and powered off for charging.

Clear all paring info: Long press the Power On/Off button and Volume down button for 5 seconds simultaneously

Charging: In case of using USB charging line for charging, connect the headphone with the USB power. During the charging, LED turns red. After charging, the LED turns blue.

#### How to wear



Hang the headphone behind your neck with both speakers clinging to your cheekbones. After that, adjust it until you feel comfortable.

### **Specifications**

Music play timeStandard: 5 hours
Standby time20 days
Charging time2hours
BT version5.0
Speaker impedance10Ω±3db
Microphone sensitivity
Product weight33g
Connection distance15 meters

#### **Attentions**

- Please wear the bone-conduction speakers of the headphone at your cheekbones before your ears and do not put them in your ears.
- Listening to music for an excessive long time mayake you feel uncomfortable, please try to use a medium volume instead of a big one.
- No matter how the volume is, please do not mal it prevent you from hearing the surrounding soun Please take care when using the headphone.
- Do not throw, press the headphone or put it in the water.
- The headphone should keep away from fire, water and high voltage devices.
- It is not for children's use as it contains small parts.
- Please do not treat it as domestic waste. Insteasend it to your electrical and electronic recycling station.
- ▲ If there is any injury or property loss due to violation of the above contents, we will not bear any responsibility.